

Omni Blue

A Vital Tool for Capsulorhexis



- A capsule stainer to visualize the capsulorhexis in cataract surgery
- Excellent Visualization of capsulorhexis in eyes with mature cataract (or) narrow pupils
- Clear outline of the capsulorhexis rim while on surgery
- Reduces risk of incomplete capsulorhexis

Active Ingredient : Trypan Blue 0.6 & 0.8 mg / ml

Concentration : 0.06% & 0.08% w/v

Packaging : 1ml. Vial & Pfs, single use unit dose

Box Packaging : 05 / 10 vials & Pfs in a box



Description :

Omni Blue is a vital dye, used to visualize the Capsulorhexis during cataract surgery in the absence of red fundus reflex. It stains the anterior Lens capsule offering excellent visualization in eyes with mature cataract. Omni Blue is a sterile preparation of Trypan Blue in an isotonic, buffered solution.

Contraindications :

No contraindications have been reported till date. Care must be taken on patients known to be hypersensitive to Trypan Blue or Similar dyes.

Indications :

1. Used as an aid in ophthalmic surgery
2. Used as a staining agent
3. Detects PC rupture
4. Anterior / Posterior Capsulotomy
5. Phacoemulsification
6. Non Phaco SI Cataract surgery
7. ECCE

Storage :

Keep in cool dry place, protect from light.

Supply :

Available in 1 ml vial & Pfs single unit, unit dose in pack of Five or Ten.

Warning and Precautions :

1. Unit dose. For single use only. Discard remaining Solution after use.
2. For Intra Ocular use only.
3. Not for IM / IV injection
4. May temporarily stain hydrophilic foldable lenses. Please exercise caution when using hydrophilic lenses.

Method of Use :

Prior to injecting the dye, it is advisable to minimize the fluid in the anterior chamber. This will prevent dilution of the dye and offer better staining. Omni Blue is injected into the eye on the lens capsule till the capsule is clearly stained.



Rated **NSIC-CRISIL SE 1A**

An ISO 13485:2003
Certified Company

CE
1293

REGD. OFFICE: _____
OMNI LENS PVT.LTD. 5, "SAMRUDDHI", OPP. SAKAR-III, NAVRANGPURA, AHMEDABAD-380 014. (India)
Ph : +91-79-27544031-32-34 Fax : +91-79-27544035 E-mail : info@omnilens.in www.omnilens.in

